# RESCUE SAM

**Automated external defibrillator (AED)**  
Defibrillatore esterno automatico (DAE)

#### Defibrillator

RESCUE SAM is a semi-automatic external defibrillator designed for providing simple and direct voice prompts and indications for a straightforward rescue operation. It is lightweight and battery powered for maximum portability.

RESCUE SAM is designed to treat Ventricular Fibrillation (VF) and Fast Ventricular Tachycardia (VT).

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# RESCUE SAM

## TECHNICAL SPECIFICATIONS

**OPERATING MODE:** semi-automatic  
**WAVEFORM:** BTE (biphasic truncated exponential type); impedance compensated  
**ENERGY:** 200J fixed as new norms AHA 2010  
- adult: nominal 200 J into a 50  $\Omega$  load  
- infant/children: less than 50J using optional attenuated pediatric pads  
**SHOCK ENERGY ACCURACY:** According to IEC EN60601-2-4  
**PROTOCOL:** voice prompts and indicators guide user through protocol  
**PACEMAKER:** detection and rejection  
**CHARGE CONTROL:** automatic by software (arrhythmia detection system and charging control)  
**CHARGE TIME FROM "SHOCK ADVISED":** <8 seconds, typical with new battery  
**VOICE INSTRUCTIONS:** detailed voice messages guide responder through use of the defibrillator  
**CONTROLS:** shock button, info-button, On/Off button  
**INDICATORS:** LEDs (different colors), info-button  
**CHARGE COMPLETE INDICATOR:**  
- Voice prompts (press the shock red button)  
- Flashing red shock button  
**DISARM:** once charged, the RESCUE SAM disarms if:  
- Patient's heart rhythm changes to non-shockable rhythm, or  
- The SHOCK button is not pressed within 15 seconds after being armed, or  
- The Power ON/OFF button is pressed to turn OFF, or  
- The defibrillator pads are removed from the patient or the pads connector is disconnected  
**SHOCK DELIVERY:** shock is delivered if the SHOCK button is pressed while the defibrillator is armed  
**SHOCK DELIVERY VECTOR:** via adult defibrillator pads in the anterior-anterior (Lead II) position or via reduced-energy pediatric pads in the anterior-posterior position  
**PATIENT ISOLATION:** type BF  
**BATTERY:** 12 Volt DC, 4.2 Ah, lithium manganese dioxide, disposable long-life  
**BATTERY CAPACITY:** min.200 shocks or 4 hours of operating time continuous  
**AUTOMATED AND USER-ACTIVATED SELF-TESTS:**  
- Daily Self-tests: tests internal circuitry, waveform delivery system, battery capacity, and software.  
- Battery Insertion Test: upon battery insertion, extensive automatic self-tests and user-interactive test check device readiness.  
**DATA RECORDING AND TRANSMISSION:**  
- Dedicated interface to download data  
- Optional special battery pack for data transmission  
- Data stored: first 60 minutes of ECG and the entire incident's events and analysis decision.  
**SIZE:** 80x300x270 mm  
**WEIGHT:** 2.2 kg with battery pack installed.

Standard set:

- main unit
- disposable def pads adult
- disposable battery pack

Optionals:

- preconnected def pads adult/pediatric
- disposable def pads pediatric
- carrying bag
- wall mounted cabinet
- software with USB cable adapter for PC data management
- wall mount

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PROGETTI S.r.l.